REF **ACTIV-TIM**

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BRUGSANVISNING

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Product code: ACTIV-TIM

1. Characteristics

The wheelchair is made of a lightweight aluminum frame. It features flip-up armrests and an optional detachable upholstery (for backrest and seat) made of a breathable mesh material. The wheelchair is equipped with a quick release function, center of gravity adjustment, attendant brakes and a foldable backrest. It has a safety belt and straps, anti-tip wheels, solid 22" rear wheels and 8" front wheels. Available also in versions with 16" and 24" wheels.

The set includes: wheelchair frame with wheels, seat and backrest, safety belt, set of keys for wheelchair assembly/dismantling.

Optional accessories: anti-tip wheels, footrests with safety straps, mesh upholstery.

Elements of the wheelchair:

2. Application

The wheelchair is designed for people who have problems with unassisted moving and for patients with dysfunctions and reduced mobility. This self-propelled wheelchair can also be pushed by an accompanying person. The wheelchair is suitable for indoor and outdoor use.

3. Assembly

Step 1. Unfolding the wheelchair

Place the wheelchair on the ground. Unfold the wheelchair using the armrests. Then, press the center of the seat so that all the elements fit together.

Step 2. Backrest and seat

Lift the backrest upwards and lock it with backrest locks which are located below the hand grips. Fix the upholstery on the wheelchair.

Step 3. Footrests

Fix the footrest on the outer side of the frame. Then turn the footrest inwards until it is locked.

Step 4. Armrests

The armrests are tiltable. To raise the armrest, press the button located on the frame (under armrests) of the wheelchair.

Step 5. Height adjustment of footrests

Adjust the footrests using the screw located on the tube. Enough space should be left between the footplates and the ground as footplates set too low may cause difficulties in climbing hills and curbs. Adjust the footplates so that the angle between trunk and thighs is 90° (optimum sitting position for the user).

Step 6. Assembly of safety straps

Attach the safety straps to the footrests to prevent the user's legs from sliding down the footplates.

FOLDING

The wheelchair can be re-folded in order to be transported and stored more easily.

- Swing away the footplates and set them in vertical position.
- Detach the upholstery.
- Use the handles to lift the seat upwards until the frame of the wheelchair is folded.

- To fold the backrest, simultaneously press locking elements which are located under the hand grips at the back of the backrest and pull the backrest downwards.

PARKING BRAKE

To lock the wheels, push the lever forward until the wheel is fully blocked (separately for left and right wheel). Please, note that the parking brake only serves to lock the wheels when the wheelchair is at a complete stop.

PARKING BRAKES ADJUSTMENT

The proper functioning of the parking brake depends on its adjustment and position. It should be mounted in adequate distance from the back wheel, so that it is blocked firmly and the lever does not face excessive resistance.

- In case of excessive resistance, move the parking brake towards the front using the locking screws. Set the parking brake in a position that allows it to block the wheels easily and without excessive resistance.
- When the parking brake does not touch or lock the wheel properly, follow the above instruction by moving the handle in the opposite direction.

ATTENDANT BRAKES

The levers of the brakes are located behind the backrest, under both grips.

- In order to stop the wheelchair, squeeze the lever of the brake. To release the brake, let go of the levers. Perform this activity smoothly and with appropriate force.
- The brakes are equipped with parking locks. In order to use them, squeeze it and pull out the locking element.

4. BEFORE USE

Step 1. Checking the condition of the wheelchair

Check the condition of the brakes, construction and tyres. Check if all the components are assembled and adjusted properly.

Note! Do not use the wheelchair if any defects are noticed.

Step 2. Using parking brakes

Lock the wheels when getting in or out of the wheelchair.

Step 3. Footplates

The footplates should be set in vertical position when getting in or out of the wheelchair. Do not use the footplates as a support while getting in or out of the wheelchair, due to the risk of falling. After transferring to the wheelchair, unfold the footplates.

Note! Do not stand on the footplates due to the risk of damage.

Step 4. Getting in the wheelchair

The wheelchair should be located as close to the patient as possible. The wheelchair should touch the bed or other seat being used. If the user cannot change their seat independently, transfer the user to the wheelchair.

Step 5. Position of the user

The user should sit in an upright position and their knees should be bent at the angle of 90°.

The wheelchair is ready to use.

5. Technical data

Seat dimensions	43 / 46 / 48 cm
Overall width	61 / 64 / 66 cm
Overall depth	106 cm
Seat height from the ground	50 cm
Overall height	92 cm
Weight	10,64 - 11,18 kg
Maximum capacity	115 kg
Folded width	106 x 29 x 92 cm

6. Notes

Timago International Group does not bear responsibility of improper moving in the wheelchair, failure to comply safety regulations and misuse.

7. Cleaning and maintenance

Carry out the maintenance of the wheelchair components regularly:

- The upholstery and the cushion can be cleaned with damp (not wet) cloth and cold or warm water, without the use of detergents. Difficult stains can be cleaned with a sponge. Do not soak the upholstery too much. Dry away from heat sources.
- Plastic elements should be cleaned with damp cloth possibly with the use of appropriate detergents for plastic elements. In such case follow the instructions given by the producer and safety measures.

8. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

9. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

10. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors.

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