REF MOBIL-TIM

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BRUGSANVISNING

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USER MANUAL

Wheelchair with transport function

Product code: MOBIL-TIM

1. Characteristics

Wheelchair is made of steel frame. It features flip-up armrests for easy entry and exit for people with reduced mobility. It is equipped with 8" back and 6" front wheels, manually controlled brakes (back wheels), seat with backrest, security belt and footrest.

Elements of the wheelchair:

- 1. Armrest
- 2. Handbrakes
- 3. Safety belt
- 4. Backrest
- 5. Seat
- 6. Footrest
- 7. Rear wheel 8"
- 8. Front wheel 6"
- 9. Safety buckle

Wheelchair after folding:

2. Application

Wheelchairs are designed for people who have problems with unassisted movement and for patients with dysfunctions and reduced mobility. The wheelchair is suitable for indoor use only. The wheelchair may only be used with the assistance of an accompanying person.

The wheelchair functions as transport chair. It facilitates everyday movement. The wheelchair is especially recommended to people who have problems with mobility which are caused by: paralysis, obesity or abnormal underweight, degenerations or deformity of limbs, degenerations or joints injuries, limbs injuries or amputations, heart and blood vessel disorder and circulatory disorders.

3. Assembly

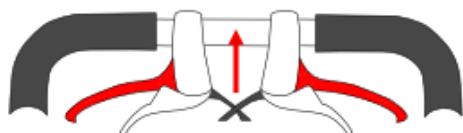
UNFOLDING/FOLDING WHEELCHAIR:

- Hold seat frame with one hand and backrest frame with another, spreading them apart till fully unfolded. Lock safety buckle located behind backseat. Pull down armrests and footrest.
- To fold wheelchair, repeat actions starting from folding armrests and footrest, then unlock safety buckle and hold backrest and seat frame, pulling them apart till fully folded.

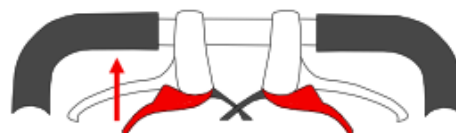
FLIP-UP FUNCTION OF ARMRESTS:

- Pull the armrest upwards with a gentle movement.
- Do not use the armrest as a support while standing up. If help is required – ask for assistance.

BRAKES

Picture 1

Picture 2

- To lock the wheels, tighten the top hand brakes (picture 1).
- To unlock the wheels, press the bottom hand brakes (picture 2).
- To stop, gently press top hand brakes.

FOOTRESTS

- Do not use the footplates as a support while getting in or out of the wheelchair due to the risk of falling over.

GETTING ONTO THE WHEELCHAIR

- It is essential to lock the brakes while getting in or out of the wheelchair, changing the user's position or getting off.

- While changing place, the wheelchair should be located as close as possible to the place of transfer. The wheelchair should touch the bed or other seat being used. If the user cannot change his/her seat independently, transfer the user to the wheelchair.
- The user should sit as deep as possible in the middle of the seat in order to avoid wheelchair instability.

4. Technical details

Overall width	51 cm
Inner width	41 cm
Overall depth	80 cm
Seat height from the ground	48 cm
Overall height	91 cm
Seat dimensions	41 x 41 cm
Weight	9,5 kg
Maximum capacity	136 kg
Frame color	white

5. Notes

Timago International Group does not bear responsibility for improper use of the wheelchair and failure to comply with safety regulations.

6. Cleaning and maintenance

Carry out the maintenance of the wheelchair components regularly:

- The seat can be cleaned with a damp cloth and cold or warm water, without the use of detergents. Difficult stains can be cleaned with a sponge.
- Plastic elements should be cleaned with damp cloth.

7. Storage and transport conditions

The product should not be exposed to direct sunlight.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that, for

warranty purposes, the proof of purchase (receipt or invoice) should be retained.

9. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

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